



Installation for MR2000 Driver

1. To Install MR2000 driver


1) Execute "SAMSUNG_USB_Driver_for_Mobile_Phones.exe", and then click "Next" button.

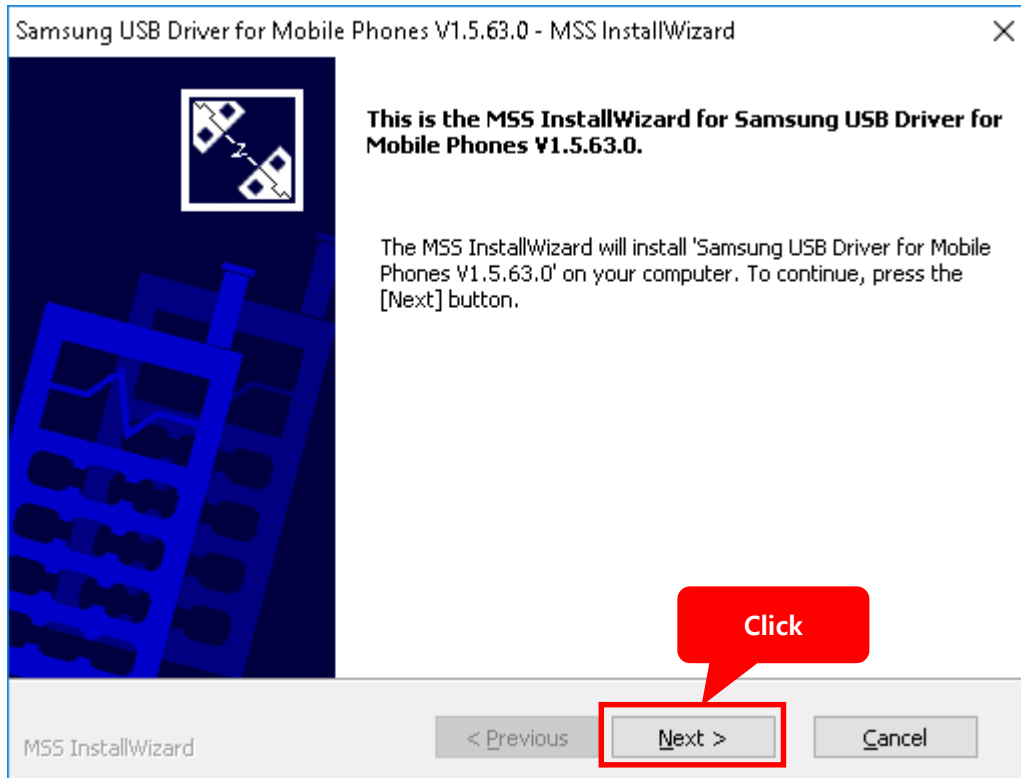


2) Click "Next" button. (Leave the language & country settings.)

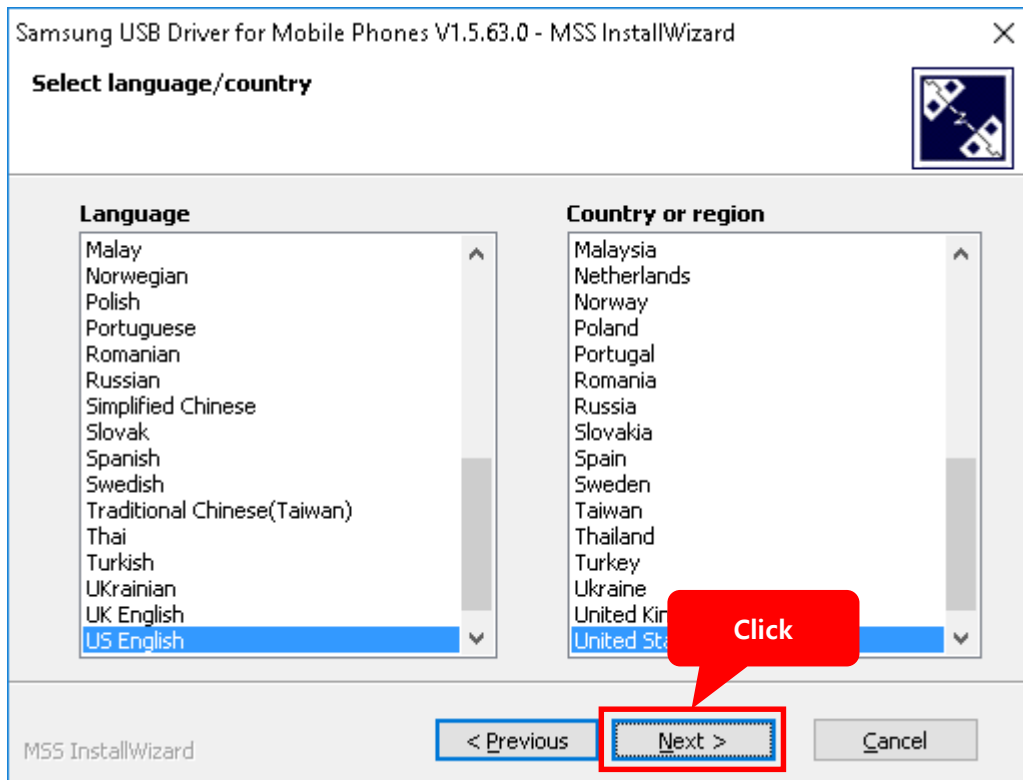


3) Proceed the driver installation with clicking "Install" button.

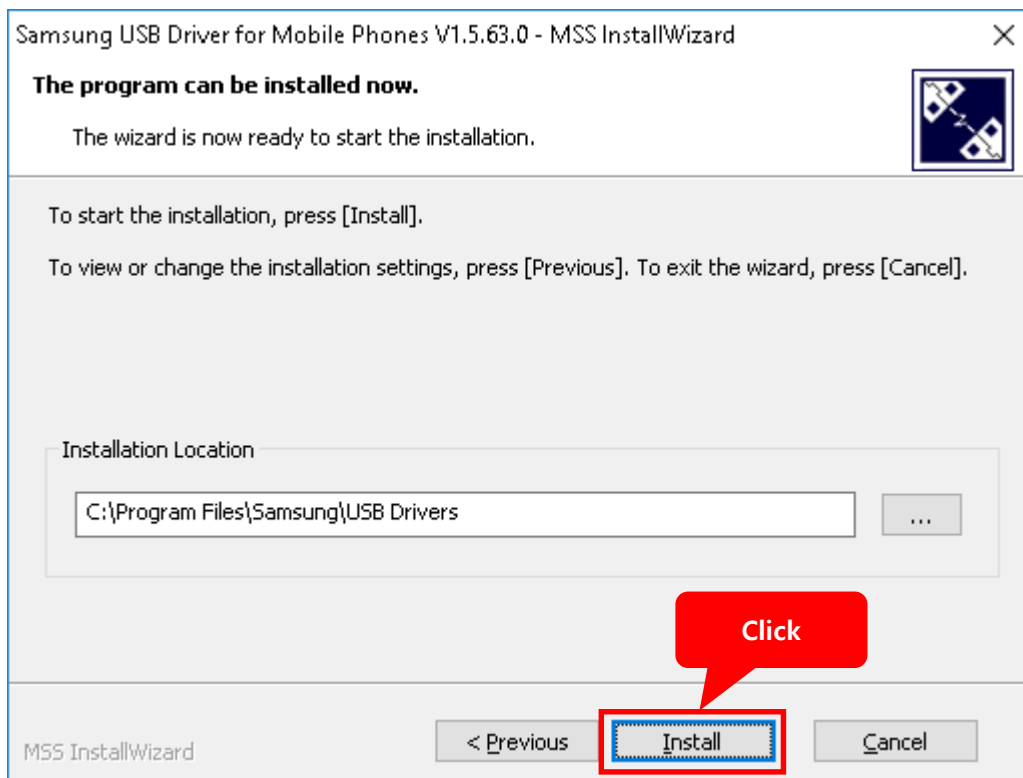

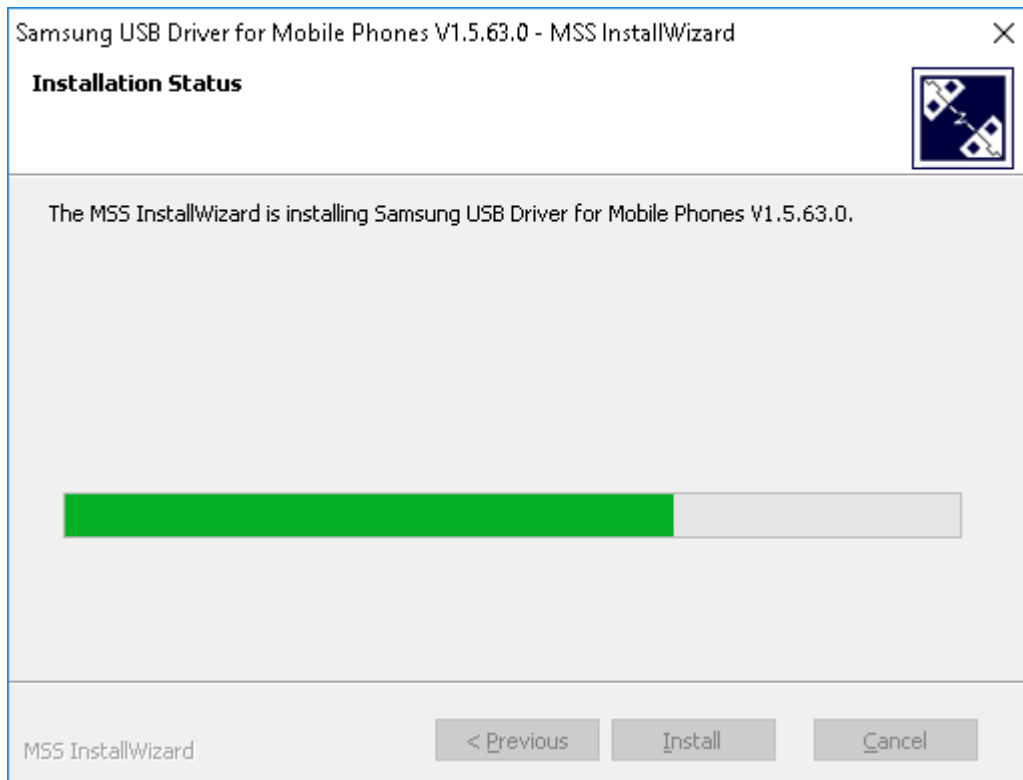


4) Exit after clicking "Finish" button.

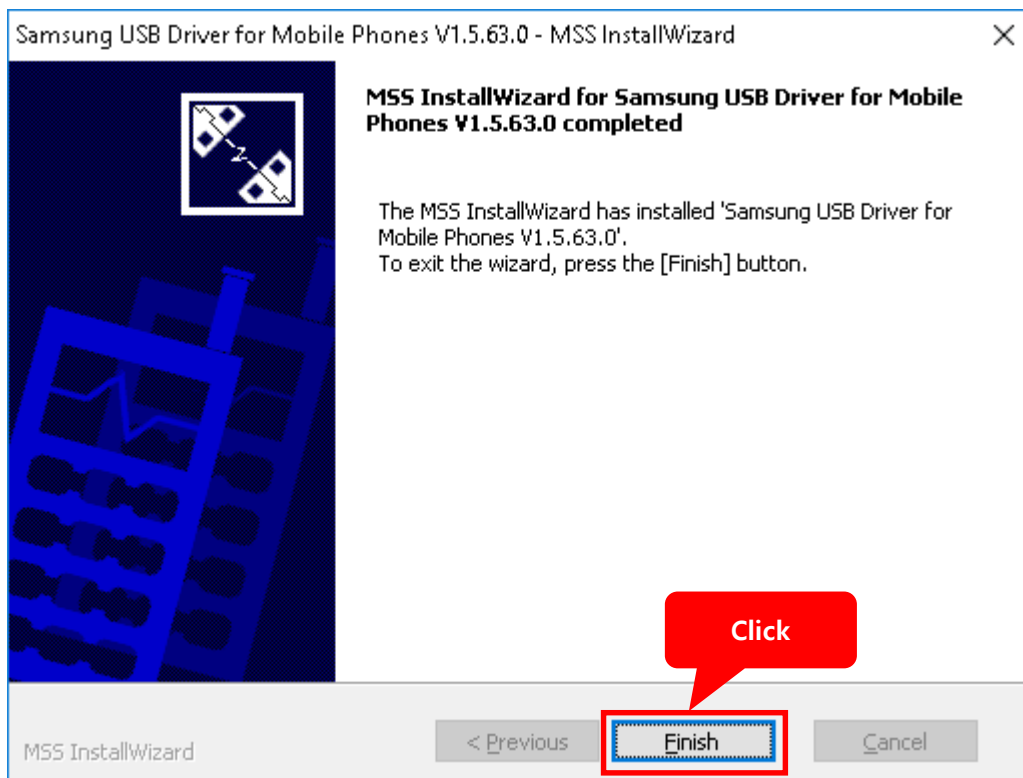


2. Installation of ADB driver for MR2000 on "Device Manager"

- 1) Connect MR2000 receiver at the USB port.
- 2) Open "Device Manager"

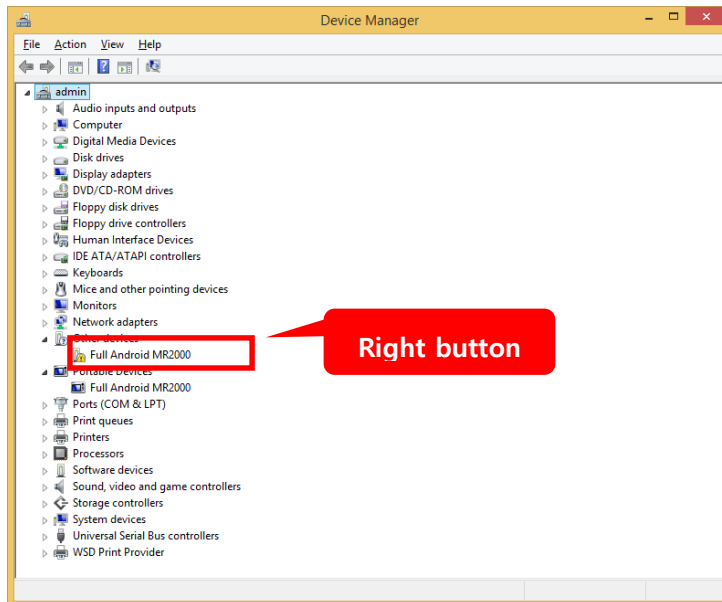


- 3) Click the right button of the mouse on "Other devices->Full Android MR2000".
- 4) Select "Update Driver Software...".

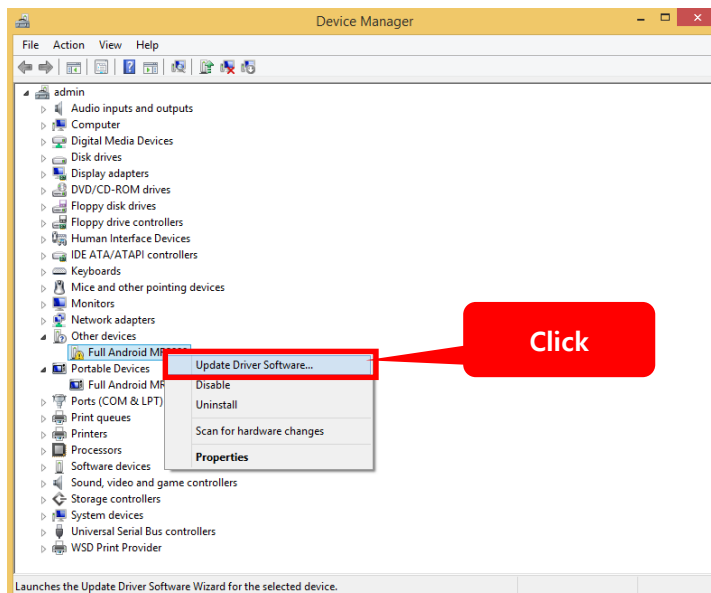


- 5) Select "Browse my computer for driver software



Locate and install driver software manually." Item on Update Driver Software page.

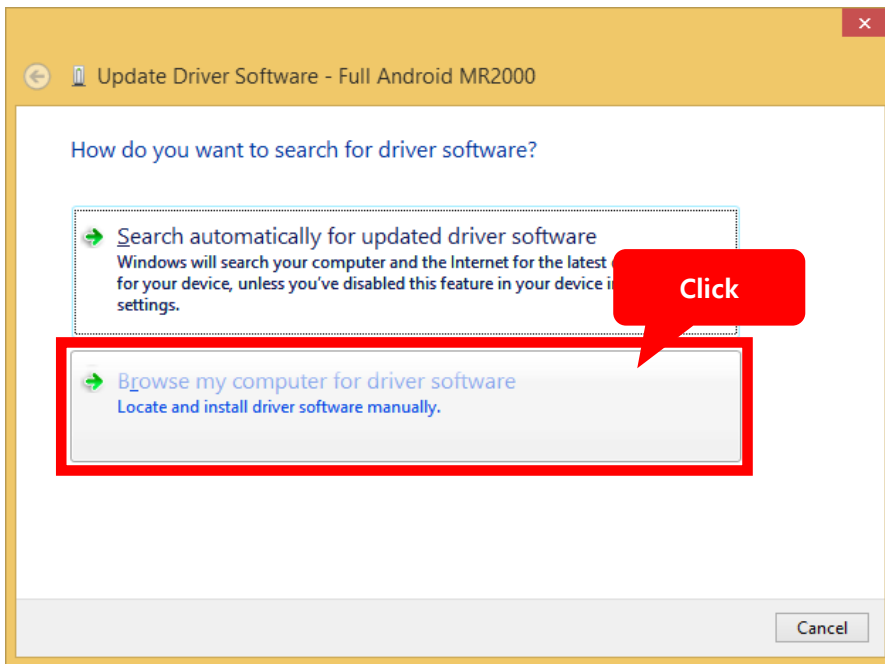

- 1) Copy and Paste `C:\ProgramData\IntroMedic\MiroView\2.0\Common\Drivers\MR2000` in the red box

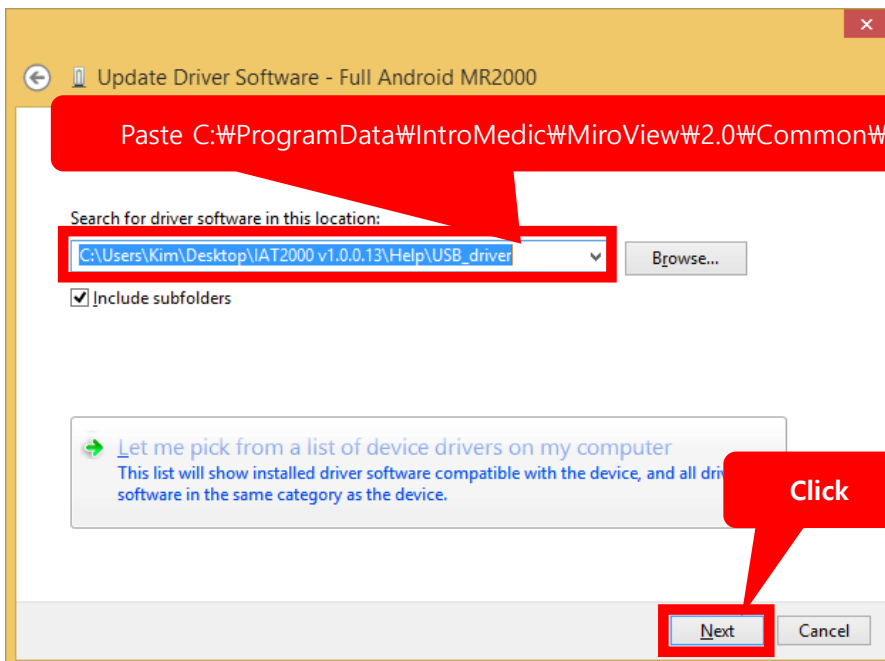

2) Proceed ADB installation by clicking "Yes" button on the below message box.

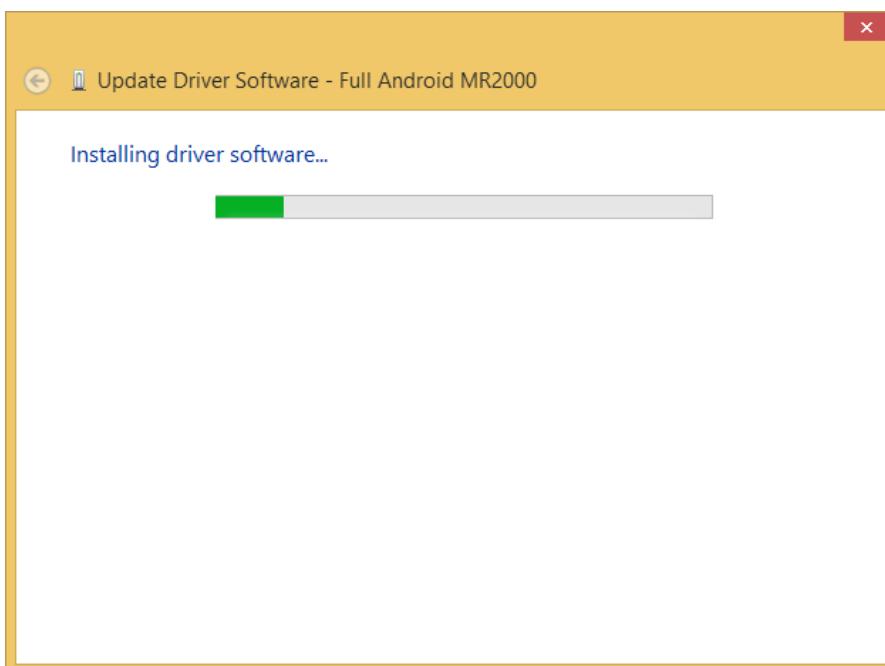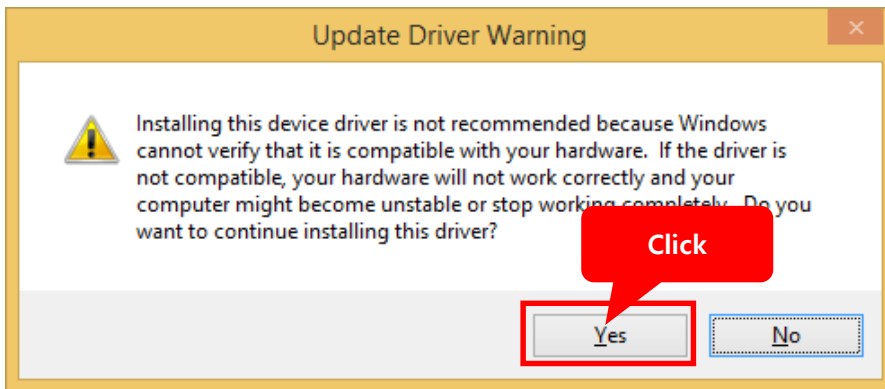

- 3) ADB installation is completed. Confirm "ADB Interface-> Android Bootloader Interface" on Device Manager after closing this page.

